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RA

USER MANUAL

X-RAY PROTECTIVE CLOTHING – OPERATOR/USER

Languages:

Deutsch

02-21

English

24-43

Art.-Code: 1240990

MAVIG

MAVIG

www.mavig.com/eifu

EN

- DE** Vielen Dank, dass Sie sich für MAVIG entschieden haben. Die Gebrauchsanweisung finden Sie unter dem Link der Überschrift.
- EN** Thank you for buying MAVIG products. The instruction for use can be found under the link in the headline.
- AR** نشكر على شراء منتجات شركة "مافيج".
يُمكنك إيجاد تعليمات الاستخدام على الرابط في العنوان.
- BG** Благодарим Ви, че избрахте MAVIG. Упътването за употреба може да намерите на линка на заглавието.
- CS** Děkujeme za zakoupení výrobku MAVIG. Návod k použití naleznete na odkazu v titulk.
- DA** Tak fordi du købte MAVIG-produkter. Brugsanvisningen kan findes under linket i overskriften.
- ES** Le damos las gracias por la compra de productos MAVIG. Las instrucciones de uso pueden encontrarse bajo el enlace del titular.
- ET** Täname, et soetasite ettevõtte MAVIG toote. Kasutusjuhend leiata pealkirja lingi alt.
- FI** Kiitos, että valitsit MAVIG-tuotteen. Käyttöohjeet löytyvät otsikon linkin takaa.
- FR** Merci d'avoir acheté les produits de MAVIG. Le mode d'emploi est disponible en cliquant sur le lien qui se trouve au niveau du titre.
- HR** Hvala što ste kupili proizvode društva MAVIG. Uputa za upotrebu nalazi se ispod poveznice u naslovu.
- HU** Köszönjük, hogy MAVIG termékeket vásárolt. A használati utasítások a címsorban lévő hivatkozásokon találhatóak.
- ID** Terima kasih Anda telah membeli produk MAVIG. Petunjuk penggunaan dapat ditemukan di bawah tautan di judul.
- IT** Grazie di aver acquistato i prodotti MAVIG. Le istruzioni per l'uso possono essere consultate sul link del titolo.
- JA** MAVIG製品をお買い上げいただきありがとうございます。使用方法の説明は、見出しのリンクの下に掲載されています。
- KO** MAVIG 제품을 구매해 주셔서 감사합니다. 사용 설명서는 헤드라인의 링크 아래에서 찾을 수 있습니다.
- LT** Dėkojame, kad įsigijote MAVIG gaminius. Naudojimo instrukciją rasite paspaudę nuorodą antraštyje.
- LV** Paldies, ka iegādājāties MAVIG produktus. Instrukciju lietošanai var atrast zem virsraksta saites.
- NL** Hartelijk dank dat u voor MAVIG gekozen heeft. De gebruiksaanwijzing is te vinden onder de link van de titel.
- NO** Takk for ditt kjøp av MAVIG-produkter. Bruksanvisningen finner du under lenken i overskriften.
- PL** Dziękujemy za zakup wyrobów MAVIG. Instrukcję użytkowania można znaleźć pod linkiem w nagłówku.
- PT** Obrigado por adquirir produtos MAVIG. As instruções de utilização encontram-se na ligação presente no título.
- RO** Vă mulțumim pentru achiziționarea produselor MAVIG. Instrucțiunile de utilizare pot fi găsite accesând linkul din antet.
- RU** Благодарим вас за приобретение изделий производства MAVIG. Инструкцию по эксплуатации можно открыть под ссылкой в заголовке.
- SL** Zahvaljujemo se vam za nakup izdelkov MAVIG. Navodila za uporabo so na voljo na povezavi v naslovni vrstici.
- SV** Tack för att du köpt en MAVIG-produkt. Bruksanvisningen kan hittas under länken i rubriken.
- TR** MAVIG ürünlerini satın aldığınız için teşekkür ederiz. Kullanım kılavuzunu başlıktaki linkin altında bulabilirsiniz.
- VI** Cảm ơn bạn đã mua sản phẩm của MAVIG. Bạn có thể xem hướng dẫn sử dụng và sách hướng dẫn lắp đặt ở liên kết trong tiêu đề.
- ZH** 感谢您购买 MAVIG 产品。如需查看使用说明，可点击标题下的链接。

Dear Customer,

Thank you for purchasing a MAVIG product. Please read this document carefully and follow all directions, warnings, and notices.

Feel free to contact us with any questions or comments on our products.

Contact Information

For questions or comments, please refer to the back of this manual for your nearest MAVIG contact.

Disclaimers

The content of this manual is for information purposes only. The products described are subject to change without notice in the course of continuous development. MAVIG only assumes warranty for this manual and the products within the scope of the manufacturer's liability.

Changes

MAVIG products are under continuous development. MAVIG reserves the right to modify the manual, packaging, equipment or technical specifications, and scope of delivery without prior notice.

Translation

The master document is in German. In case of translation, precedence remains with the German version.

Copyright and Trademarks

All rights reserved. These instructions/manuals are protected by copyright law. MAVIG and Novalite are registered trademarks. All other trademarks mentioned in the manuals are the property of the corresponding manufacturer.

Table of Contents

| | | |
|-------|--|----|
| 1.0 | Use and Availability of the Instructions for Use | 27 |
| 2.0 | Warranty | 27 |
| 3.0 | Symbols and Definitions | 28 |
| 4.0 | Intended Use | 30 |
| 5.0 | Specifications | 34 |
| 5.1 | Conditions of Transport & Storage | 34 |
| 5.2 | Identification of the Product | 34 |
| 5.3 | Standards | 35 |
| 5.4 | Product Description | 36 |
| 5.4.1 | Degree of Protection (and Area Weight) | 36 |
| 5.4.2 | Sizing | 38 |
| 5.5 | Replacement Parts | 39 |
| 5.6 | Manufacturer & Contact Information | 39 |
| 5.7 | Repair | 39 |
| 5.8 | Shipment / Return Shipment | 39 |
| 6.0 | Scope of Delivery | 40 |
| 7.0 | Inspection | 41 |
| 8.0 | Cleaning & Disinfection | 42 |
| 8.1 | Cleaning | 42 |
| 8.2 | Disinfection | 42 |
| 9.0 | Disposal | 43 |

1.0 Use and Availability of the Instructions for Use

These instructions for use cover the following products:

Personal Protection Equipment (PPE) - Radiation Protective Clothing

| | | |
|--------------------|-------------|---|
| Model range | RA631 | All-round protective vest and skirt |
| | RA632 | All-round protective coat |
| | RA636 | All-round protective coat |
| | RA640 | Front protective apron |
| | RA641 | Front protective apron |
| | RA650 | Front protective apron |
| | RA660 | Front protective apron |
| | RA614 | Thyroid and sternum protection |
| | RA610 | Radiation protective headband |
| | RA611 | Radiation protective cap with breathable insert |
| | RA612 | Radiation protective cap |
| | RA637 | Radiation protective bolero |
| | RASLF/RASLR | Radiation protective sleeves |

This manual is part of the product and therefore should be kept in the immediate vicinity of the product, so that important information and safety instructions for use can be looked up at all times.

Do not give the product to a third party without a valid manual.

2.0 Warranty

Our general terms of business apply. These should be available to the operator latest after completion of the sales contract.

MAVIG guarantees the safety and reliability of the product solely on the following basis:

- The product is used solely for its intended purpose and its operation complies with the information provided in these instructions for use.
- Only original authorised replacement parts or accessories as specified by MAVIG are used. The use of unauthorised parts holds unknown risks and must be avoided at all times.
- No unauthorised changes are made to the product.
- The product is regularly inspected and serviced at the specified times.

The workmanship and materials used have a 24-month manufacturer's warranty.

3.0 Symbols and Definitions

Information in these instructions that is particularly important is highlighted below. Safety warnings are mandatory and must be adhered to in order to prevent personal injury or damage to equipment.

WARNING

Indicates that the information is a warning. Warnings refer to circumstances which could result in injury or death of the patient or operator/user. Read and understand all warnings before using the product.

CAUTION

Indicates that the information is a caution. Cautions advise you of circumstances that could result in damage to the product or improper functioning of the product. Read and understand the cautions before using the product.

WARNING - Magnetic field

Indicates that the information warns about a magnetic field. Warnings refer to circumstances which could result in injury or death of the patient or operator/user. Read and understand all warnings before using the product.

No Persons with Pacemakers

This symbol indicates that this product must not be used by persons with pacemakers (minimum distance > 20 cm).

NOTE:

Provides usage tips and useful information.

COMPLY WITH MANUAL

This symbol represents to consult accompanying documents.

CE SYMBOL

With this symbol MAVIG declares that the products comply with the relevant regulations set forth in the applicable European Directives.

ARTICLE NUMBER

Shows the manufacturer's article number.

SERIAL NUMBER

Shows the manufacturer's serial number.

BATCH NUMBER

Shows the manufacturer's batch number.

MANUFACTURER

Shows the address and the manufacturer or distributor of the PPE.

DATE OF MANUFACTURE

Indicates the date of manufacture of the product.

3.0 Symbols and Definitions

TEMPERATURE LIMIT

Specifies the temperature limit, which the product can be exposed to safely.

STORE PROTECTED FROM SUNLIGHT

Indicates that the product should not be stored in direct sunlight.

KEEP DRY

Indicates that the product should be stored in a dry place.

STORAGE PERIOD

Indicates the maximum storage time in the packaging.

PROTECTION AGAINST IONISING RADIATION

Indicates that the product provides protection against ionising radiation.

DO NOT WASH

Indicates that the product must not be washed in the washing machine.

DO NOT BLEACH

Indicates that the product must not be bleached.

DO NOT IRON

Indicates that the product must not be ironed.

DO NOT TUMBLE DRY

Indicates that the product must not be tumble-dried.


DO NOT DRY CLEAN

Indicates that the product must not be dry-cleaned.

DO NOT WET CLEAN

Indicates that the product must not be wet-cleaned.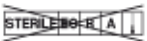

NON-STERILISABLE

Indicates that the product cannot be sterilized with ethylene oxide, radiation, aseptic processing techniques, steam, or dry heat.

4.0 Intended Use

Radiation protection clothing is mandatory as personal protective equipment for operator/users of systems that use X-rays for diagnostics and protects - in the X-ray tube voltage range of 50 - 150 kV (StandardLite) or 50 - 110 kV (NovaLite / OmniLite) - against ionising radiation.

Make sure to put on your radiation protective clothing correctly. This is essential to ensure optimum radiation protection in accordance with legal requirements.

Please pay attention that the radiation protective apron covers your front area from neck to knees. An all-round apron must also cover your back area as well as your sides starting from approx. 10 cm under the armpit down to the knees.

In addition, please ensure when choosing radiation protective clothing that the width of the front protective apron or the width of the front area of the all-round protective apron covers 60% of your body circumference. Refer to the circumference of the chest, waist, or hip, depending on which of the three dimensions indicates the greatest size.

NOTE: *If these conditions are not met, add a thyroid shield or sleeve, or change the size or length.*

WARNING - Insufficient Protective Area

Make sure the shoulder seam sits correctly on your shoulder and does not fall forward. If this is the case, the upper shoulder area up to the shoulder seam is not protected with the full front lead equivalent since the back of the radiation protective clothing usually has a lower or no lead equivalent. This leads to increased radiation exposure.

The thyroid shield should cover the front half of your neck, including the thyroid gland, and reach down to the neckline of the radiation protection apron.

Inspect products for visible damage before use. For this purpose see also "Inspection" (Chapter 7.0).

Model RA637 – Radiation Protective Bolero

NOTE: *Model RA637 is solely intended for combination with MAVIG front or all-round protective aprons.*

Put on the bolero so that it can be closed in the front. Close the hook and burr closure. After that, put on your preferred model over the bolero.

4.0 Intended Use

Models RASLF / RASLR – Radiation Protective Sleeves

NOTE: *Model RASLF is solely intended for combination with MAVIG front protective aprons (Models RA640, RA641, RA660). Model RASLR is solely intended for combination with MAVIG all-round protective aprons (Modelle RA631, RA632). The size of the sleeve has to match the size of the apron.*

Attach the detachable sleeves to the designated side of the radiation protective apron using the existing snap fasteners. The longer seam of the sleeve should represent an extension of the shoulder seam when fastened. The position of the snap fasteners automatically ensures an overlap between the sleeve and the radiation protective apron.

WARNING - Insufficient Protective Area

If an incorrect sleeve is chosen (model or size) or if the sleeves is not attached properly or to the correct side of the apron, sufficient and continuous protection cannot be ensured. This can lead to increased radiation exposure.

Models RA610, RA611, RA612 - Head Wear

Place the radiation protective head wear on your head. Adjust the head wear to your individual head circumference using the stopper.

Model RA631 - All-round Protective Vest and Skirt

Put on the skirt so that it wraps around the back of your entire waist. Pull on both skirt panels so that the stretch insert is stretched. When closed, this procedure results in a better weight distribution of the radiation protective clothing. Close the two skirt panels on top of each other.

Make sure that the end of the outer skirt panel reaches at least the marked safety zone (section in the colour "Titan", limited by two vertical seams in the outer material). If this is not the case, or if there is an overlap beyond the safety zone, choose a different skirt size.

WARNING - Insufficient Protective Area

If the end of the outer skirt panel does not reach the marked safety zone (section in the colour "Titan", limited by two vertical seams in the outer material), the non-overlapping area is not protected with the specified lead equivalent. This leads to increased radiation exposure.

Close both snap fasteners, which are located above the stretch insert and at knee level. Both closures can each be individually adjusted to the length needed.

Put the vest on so that it can be closed in the front. Close the two front vest panels on top of each other.

RA631 unisex vest only (type "W"): Make sure that the end of the outer vest panel reaches at least the marked safety zone (section in the colour "Titan", limited by two vertical seams in the outer material). If this is not the case, or if there is an overlap beyond the safety zone, choose a different vest size.

4.0 Intended Use

Model RA631 - All-round Protective Vest and Skirt

WARNING - Insufficient Protective Area, Unisex Vest (type "W")

If the end of the outer vest panel does not extend to the marked safety zone (section in the colour "Titan", limited by two vertical seams in the outer material), the non-overlapping area is not protected with the specified lead equivalent. This leads to increased radiation exposure.

Make sure the vest and skirt overlap at the front and back at all times. In particular, the entire stretch insert of the skirt must be covered by the vest. For this reason, the back of the vest is longer to ensure continuous radiation protection.

WARNING - Insufficient Protective Area

If the vest does not overlap the skirt, especially the stretch insert, the area between vest and skirt is not protected. This leads to increased radiation exposure. Please pay attention for any combination of existing and newly purchased PPE products (e.g. skirt and vest). This also implies a combination of MAVIG and non-MAVIG PPE products.

Model RA632 - All-round Protective Coat

Put the coat on so that it can be closed in the front. Close the two front panels of the coat on top of each other.

Make sure that the end of the outer coat panel reaches at least the marked safety zone (section in the colour "Titan", limited by two vertical seams in the outer material). If this is not the case, or if there is an overlap beyond the safety zone, choose a different coat size.

WARNING - Insufficient Protective Area

If the end of the outer coat panel does not extend to the marked safety zone (section in the colour "Titan", limited by two vertical seams in the outer material), the non-overlapping area is not protected with the specified lead equivalent. This leads to increased radiation exposure.

With slightly raised shoulders, close the stretch belt so that it noticeably distributes part of the weight of the coat on your hips. Then close the snap buckle on the stretch belt and the snap buckle on the coat (knee). Both can each be individually adjusted to the length needed. The stretch belt can be fastened on one or both sides of the apron using the snap fasteners to ensure that the belt sits at the appropriate and ergonomically best height for you.

4.0 Intended Use

Model RA636 - All-round Protective Coat

Put the coat on so that it can be closed in the front. Close the two front panels of the coat on top of each other.

With slightly raised shoulders, close the stretch belt so that it noticeably distributes part of the weight of the coat on your hips. Then close the snap buckle on the stretch belt. It can be individually adjusted to the length needed. The stretch belt can be fastened on one or both sides of the apron using the snap fasteners to ensure that the belt sits at the appropriate and ergonomically best height for you.

Model RA640 - Front Protective Apron

Put the front protective apron on so that your entire front from the neck to the knees is covered by the apron and with that by radiation protective material. Close the respective closure of the front apron:

- Designed for closing in the back: snap fasteners, crossbands (crossed in the back), 2 snap buckles
- Designed for closing in the front: strap and snap buckle (crossed in the back)

Model RA641 - Front Protective Apron

Put the front protective apron on so that your entire front from the neck to the knees is covered by the apron and with that by radiation protective material. Close the hook and burr fastener so that part of the weight of the apron is noticeably transferred to your hips by the inner belt. Then close the respective second closing mechanism:

- Designed for closing in the back: 2 snap buckles
- Designed for closing in the front: strap and snap buckle (crossed in the back)

NOTE: *The snap fastener closure on the wide side wings of the apron only has to be closed when working in a standing position. When working sitting down, it should remain open.*

This front protective apron has been developed for working while standing and sitting. For this purpose, it has a removable gonad protection on the inside, which hangs protectively in front of the gonads in the sitting position.

Make sure the gonad protection is attached when working while sitting.

WARNING - Insufficient Protective Area

When sitting without gonad protection sufficient protection of the gonads cannot be ensured. This can lead to increased radiation exposure.

NOTE: *The removable gonad protection is not allowed to be used individually, for example as gonad protection for patients.*

4.0 Intended Use

Model RA650 - Front Protective Apron

Put the front protective apron on so that your entire front from the neck to the knees is covered by the apron and with that by radiation protective material. Close the hook and burr fastener (crossed in the back).

Model RA660 - Front Protective Apron

Put the front protective apron on so that your entire front from the neck to the knees is covered by the apron and with that by radiation protective material. Close the hook and burr fastener (crossed in the back).

RA614 - Thyroid and Sternum Protection

Put on the thyroid shield so that it covers the front of your neck including the thyroid gland and overlaps the apron. Close the thyroid shield in the neck.

WARNING - Insufficient Protective Area

An overlap of thyroid shield and radiation protective clothing is necessary to ensure continuous radiation protection. Otherwise radiation exposure is increased.

Using the snap fasteners on the sides, the thyroid shield can be attached to the radiation protective clothing.

CAUTION

Please note that the magnetic closure of the thyroid shield must be closed in a particular way. Always close the magnetic closure so that - from your point of view - the right magnet is placed over the left magnet. Otherwise, the closure may not hold.

WARNING – Magnetic Field

The magnetic closure of the thyroid shield contains permanent magnets that create a magnetic field. This can lead to malfunctioning of implanted medical devices.


WARNING – No Persons with Pacemakers

The version of the thyroid protection with magnetic closure contains permanent magnets that generate a magnetic field. The thyroid protection is therefore not to be used by medical personnel with pacemakers or on patients with pacemakers.

5.0 Specifications

5.1 Conditions of Transport & Storage

Follow the storage conditions and the storage period on the packaging.

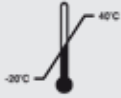Store between
-20°C and + 40°C

Protect from heat

Protect from moisture

Maximum storage duration

Use MAVIG special coat hangers or suitable MAVIG storage systems for storing protective aprons (see MAVIG catalogue). Radiation protective clothing should not be folded or allowed to hang over edges, and must be stored away from sources of heat.

Ensure that X-ray protective clothing is kept, stored, or used only within the standard temperature and humidity range for the home and work environment.

5.2 Identification of the Product

The production label contains the following information: model number (REF), description, colour, size, material data incl. tested X-ray tube voltage range, lead equivalent(s), surface weight(s), CE marking, serial or batch number, manufacturer, date of manufacture (YYYYMMDD), cleaning/disinfection instruction, care label, corresponding standards, and letter(s) designating the type of radiation protective apron.

The letter(s) designating the type of radiation protective apron have the following meaning:

- L (light): The lead equivalent of light-duty protective aprons is not less than 0.25 mmPb over the entire protective area.
- H (heavy): The lead equivalent of heavy-duty protective aprons is not less than 0.35 mmPb over the entire protective area.
- LC (light, closed): The lead equivalent of light-duty closed protective aprons is not less than 0.25 mmPb over the entire protective area.
- HC (heavy, closed): The lead equivalent of heavy-duty closed radiation protective aprons is not less than 0.35 mmPb for the protective front area and not less than 0.25 mmPb for the protective back area.

Front protection (RA640, RA641, RA650, RA660) can be designed as a light- or heavy-duty radiation protective apron.

All-round protection (RA631, RA632, RA636) can be designed as a light- or heavy-duty closed radiation protective apron.

In addition, a second label is available for your own markings.

5.0 Specifications

5.3 Standards

CE marking

CE 0302

With this symbol MAVIG declares that the products comply with the provisions in the applicable EU directives. Conformity is indicated on the label by the CE mark with the four-digit identification number of the notified body.

MAVIG declares that the products are in conformity with IEC 61331-1:2014, DIN EN 61331-1:2016, IEC 61331-3:2014, DIN EN 61331-3:2016, the German X-ray Ordinance (RöV), and the Regulation (EU) 2016/425 of the European Parliament and of the Council of 9 March 2016 on personal protective equipment and repealing Council Directive 89/686/EEC (published in the Official Journal of the European Union, L81, 31 March 2016). MAVIG is DIN EN ISO 13485:2016 certified.

The EC-type examination was conducted by:

Identification number of the notified body:

0302
ANCCP Certification Agency S.r.l.
Via dello Struggino, 6
57121 Livorno LI, Italy

The Declaration(s) of Conformity of personal protective equipment can be found under the following link: www.mavig.com/company/conformity

5.0 Specifications

5.4 Product Description

Personal protective equipment - radiation protective clothing - is mandatory for operators/users of systems that use X-rays for diagnostics, and thereby serves to protect against ionizing radiation.

5.4.1 Degree of Protection (and Area Weight)

According to test certificates number 6250517-1 and 6250517-2 of the Physikalisch-Technische Bundesanstalt and 2019040214_2 of the National Physical Laboratory, the radiation protective materials achieve the following absorption values for the indicated X-ray tube voltages according to EN 61331-1: 2014:

| Material: Standard Lead | | | | | | | | | | |
|--------------------------------|-----------------|--------|--------|--------|--------|-------|--------|--------|--------|--------|
| Weight | Lead equivalent | 50 kV | 60 kV | 70 kV | 80 kV | 90 kV | 100 kV | 110 kV | 120 kV | 150 kV |
| (3.30 kg/m ²) | 0.25 mmPb | 99 % | 97 % | 96 % | 93 % | 91 % | 88 % | 87 % | 84 % | 80 % |
| (4.60 kg/m ²) | 0.35 mmPb | > 99 % | 99 % | 98 % | 96 % | 95 % | 92 % | 92 % | 90 % | 87 % |
| (6.60 kg/m ²) | 0.50 mmPb | > 99 % | > 99 % | 99 % | 98 % | 97 % | 96 % | 95 % | 94 % | 92 % |
| (13.2 kg/m ²) | 1.00 mmPb | > 99 % | > 99 % | > 99 % | > 99 % | 99 % | 99 % | 99 % | 99 % | 98 % |


| Material: Novalite | | | | | | | |
|---------------------------|-----------------|--------|-------|-------|--------|--------|--|
| Weight | Lead equivalent | 50 kV | 70 kV | 90 kV | 110 kV | 150 kV | |
| (3.05 kg/m ²) | 0.25 mmPb | 99 % | 96 % | 91 % | 87 % | - | |
| (4.30 kg/m ²) | 0.35 mmPb | > 99 % | 98 % | 95 % | 92 % | - | |
| (6.10 kg/m ²) | 0.50 mmPb | > 99 % | 99 % | 97 % | 96 % | - | |

| Material: Leadfree | | | | | | | |
|---------------------------|-----------------|--------|-------|-------|--------|--------|--|
| Weight | Lead equivalent | 50 kV | 70 kV | 90 kV | 110 kV | 150 kV | |
| (2.95 kg/m ²) | 0.25 mmPb | 99 % | 96 % | 91 % | 85 % | - | |
| (4.18 kg/m ²) | 0.35 mmPb | > 99 % | 98 % | 94 % | 91 % | - | |
| (5.89 kg/m ²) | 0.50 mmPb | > 99 % | 99 % | 97 % | 95 % | - | |

5.0 Specifications

5.4.2 Sizing

When choosing radiation protective clothing, it must be ensured that the width of the front protective apron or the width of the front area of the all-round protective apron covers 60% of the body circumference. The circumference of the chest, waist, or hip, depending on which of the three dimensions indicates the greatest size, should be used for this purpose.

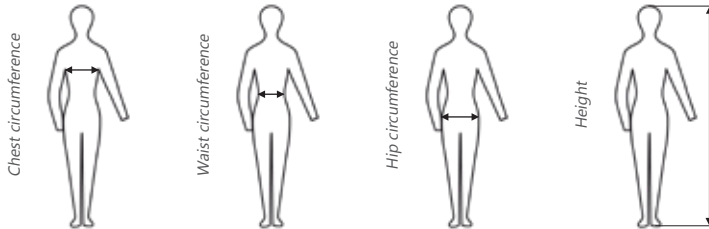All-round Protection RA631 (dimensions in cm)

| Vest & Skirt size | XS | S | M | L | XL | XXL | 3XL |
|-----------------------|-----------|-----------|-----------|-----------|-----------|-----------|-----------|
| <i>Body dimension</i> | | | | | | | |
| Chest girth | ≤ 84 | 85 - 95 | 96 - 106 | 107 - 117 | 118 - 128 | 129 - 139 | 140 - 150 |
| Waist/hip girth | ≤ 86 | 87 - 97 | 98 - 107 | 108 - 118 | 119 - 129 | 130 - 140 | 141 - 151 |
| Skirt length | 65 | 70 | 75 | 80 | | | |
| Height | 150 - 170 | 170 - 180 | 180 - 190 | 190 - 200 | | | |

All-round Protection RA632, RA636 (dimensions in cm)

| Coat size | XS | S | M | L | XL | XXL | 3XL |
|-----------------------|-----------|-----------|-----------|-----------|-----------|-----------|-----------|
| <i>Body dimension</i> | | | | | | | |
| Chest girth | ≤ 84 | 85 - 95 | 96 - 106 | 107 - 117 | 118 - 128 | 129 - 139 | 140 - 150 |
| Waist/hip girth | ≤ 98 | 99 - 108 | 109 - 119 | 120 - 130 | 131 - 141 | 142 - 152 | 153 - 163 |
| Coat length | 90 | 100 | 110 | 120 | 130 | 140 | |
| Height | 150 - 160 | 160 - 170 | 170 - 180 | 175 - 190 | 185 - 195 | 195 - 205 | |

Front Protection RA640, RA641, RA650 and RA660 (dimensions in cm)

| Apron size | XS | S | M | L | XL | XXL | 3XL |
|-----------------------|-----------|-----------|-----------|-----------|-----------|-----------|-----------|
| <i>Body dimension</i> | | | | | | | |
| Chest girth | ≤ 84 | 85 - 95 | 96 - 106 | 107 - 117 | 118 - 128 | 129 - 139 | 140 - 150 |
| Waist/hip girth | ≤ 91 | 92 - 102 | 103 - 112 | 113 - 123 | 124 - 134 | 135 - 145 | 146 - 156 |
| Apron length | 90 | 100 | 110 | 120 | 130 | 140 | |
| Height | 150 - 160 | 160 - 170 | 170 - 180 | 175 - 190 | 185 - 195 | 195 - 205 | |

5.0 Specifications

5.4.2 Sizing

NOTE: *Please note that the above-mentioned correlation of the skirt, coat, or apron length to body size applies to a standing work position. For working while sitting, the apron should be at least 10 cm longer in order to cover your knees.*

The skirt, coat, or apron length is subject to a tolerance of ± 2 cm.

CAUTION

Please note that the protective area of the front (area without pinholes) ends at the seams. This also applies to the sewn-on sleeves. In the case of the all-round protection models RA631 and RA632, the marked safety zone (section in the colour "Titan", limited by two vertical seams in the outer material) still belongs to the protective area of the front.

The length of the protective area does not exactly equal the specified skirt, coat, or apron length.

5.5 Replacement Parts

For an up-to-date list of replacement parts, please contact us at:

Phone +49 (0) 89 / 420 96 - 0 e-mail info@mavig.com

5.6 Manufacturer and Contact Information

| | | |
|-------------------|--------|--|
| MAVIG GmbH | Phone | +49 (0) 89 / 420 96 - 0 |
| Stahlgruberring 5 | Fax | +49 (0) 89 / 420 96 - 200 |
| 81829 Munich | e-Mail | info@mavig.com |
| Germany | Web | www.mavig.com |

5.7 Repair

A repair service is available through our factory in Munich. Our service hotline will gladly provide you with all the information needed (see Chapter 7.0 "Inspection").

We do not accept any warranty for repairs not performed by MAVIG.

5.8 Shipment / Return Shipment

MAVIG radiation protective clothing must not be subjected to mechanical stress during shipment. It is not allowed to fold, bent, or stack radiation protective clothing. It is recommended that the products are loosely rolled and packed properly supported in a strong shipping box of a suitable size.

To return goods to our factory in Munich, please contact our service hotline [in advance](#) (see Chapter 5.6 "Manufacturer and Contact Information"), so that we can send you the necessary return documents (including decontamination certificate).

NOTE: *It is prohibited by law to send potentially contaminated goods. Any returns must therefore be properly cleaned and decontaminated. Please attach the completed decontamination certificate on the outside of the packaging.*

6.0 Scope of Delivery

Each radiation protective product comes with an instruction manual. All radiation protective aprons also include a MAVIG special hanger that can be used to safely store the radiation protective apron.

RA631 Upon delivery, the stretch insert is only attached on one side of the skirt. Before using the radiation protective skirt, the second zipper has to be closed.

To do so, loosen the hook and burr between the stretch insert and the skirt. Close the second zipper. In the end, attach the stretch insert on the skirt via the hook and burr at regular intervals.

When replacing the stretch insert, proceed as described above.

RA632,
RA636 The included stretch belt is located in the inlay of the delivery box. Attach the stretch belt to your radiation protective coat on one or both sides at the desired height.

When replacing the belt, proceed as described above.

RA640 For the crossband closure, the burr part of the hook and burr can be replaced. To replace the elastic bands, open the snap fasteners on the back wings.

RA641 The hook and burr closure, which closes the inner stretch belt in the back, can be replaced. To do so, open the zippers on the sides of the apron.

The gonad protection can be replaced. To do so, open the hook and burr fastener on the upper edge of the protection.

RA660 The hook and burr closure on the elastic bands that cross over the back can be replaced. To do so, push back the fabric tunnel around the elastic band until you see the zipper. Then open the zipper at the end of the elastic band.

7.0 Inspection

**CAUTION**

Prior to each use of the product, check for visible damage. Do not use a damaged product!

To ensure the radiation protective clothing still provides complete protection, it should be checked regularly, at least once every 3 months. Tears, clearly visible kinks, protrusions, and uneven areas are all indications of a defect.

We recommend that the radiation protective clothing is inspected according to the standard DIN 6857-2:2021. This involves a fluoroscopic examination with X-rays at regular intervals, in addition to regular visual and tactile tests.

Please contact us if you suspect a defect:

Phone +49 (0) 89 / 420 96 - 0
Fax +49 (0) 89 / 420 96 - 200
e-Mail info@mavig.com

8.0 Cleaning and Disinfection

NOTE: *Lacking hygiene can lead to infections of patients, operator/users, or third parties. Therefore clean and disinfect according to the applicable statutory provisions.*

The following markings apply to radiation protective clothing:

Do not
wash

Do not
bleach

Do not
iron


Do not
tumble dry

Do not
dry clean

Do not
wet clean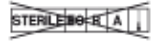

Non-
sterilisable

8.1 Cleaning

WARNING – Damage to the Product

In order to avoid damage to the product, no aggressive, abrasive or corrosive agents should be used (e.g. diluent, acetone, scouring agents).

Fresh stains can usually be removed from the radiation protective clothing with soapy water of approx. 20°C and a soft cloth. For stubborn stains, we recommend a 10% solution of sodium thiosulphate in water. For fluids containing proteins (such as blood), use cold water. Please make sure that the cloth is damp and not wet.

8.2 Disinfection

WARNING – Damaging to the Product

Do not use alcohol-based disinfectants, pure alcohol (e.g. ethanol and isopropanol), or oxidising as well as active chlorine or salt-containing disinfectants as they attack the surface of the outer material more strongly than disinfectants based on other chemicals.

The product should always be cleaned as described in Chapter 8.1 "Cleaning" prior to disinfection.

Avoid applying too much pressure while disinfecting. To maintain the properties of our radiation protective clothing, we recommend the use of the following disinfectants. Please observe the instructions below:

- The tests carried out at our company do not correspond to any certified or standardized test procedures, so that we cannot issue a release, but only a recommendation.
- Since we have no influence on the production as well as the raw materials of the disinfectant, we have to decline any guarantee on the resistance of our protective clothes to the disinfectant.
- The effectiveness of the surface disinfectant has not been considered. The instructions for use of the disinfectant must be adhered to.
- For the reasons mentioned above MAVIG excludes any warranty or liability for subsequent damages.

8.0 Cleaning and Disinfection

8.2 Disinfection

| Disinfectant | Manufacturer | Effective base |
|---------------------------|---------------------|---|
| Microbac forte | BODE Chemie GmbH | Quaternary compounds, alkylamine derivative |
| Kohrsolin FF | BODE Chemie GmbH | Aldehyde, quaternary compounds |
| Antifect FF | Schülke & Mayr GmbH | Quaternary compound, aldehyde |
| Microzid Sensitive Wipes | Schülke & Mayr GmbH | Quaternary compounds |
| Microbac Tissues Solution | BODE Chemie | Quaternary compounds |

In general, when choosing a disinfectant, make sure it contains as few alcohols as possible. Ideally, the first 5 ingredients should not be alcohols or these should be listed with amounts of „< 5%“.

If you have any further questions about disinfectants or if your clinic does not allow the disinfectants listed above, please do not hesitate to contact us:

Phone +49 (0) 89 / 420 96 - 0

e-Mail info@mavig.com

WARNING – Product Performance and Surface Damage

The combined use of different disinfectants and/or disinfectants and cleaning agents may impair the product performance or damage surfaces of the product.

9.0 Disposal

The product must be disposed of in accordance with the nationally applicable provisions at an appropriate recycling facility.

If you have any questions please do not hesitate to contact us at/on:

Phone +49 (0) 89 / 420 96 - 0

e-Mail info@mavig.com

WORLDWIDE MAVIG OFFICES

MAVIG GmbH
Headquarters

PO Box 82 03 62
81829 Munich
Germany

Stahlgruberring 5
81829 Munich
Germany

Phone +49 (0) 89 420 96 0
Fax +49 (0) 89 420 96 200
e-Mail info@mavig.com

Nordic & Baltic Countries
MAVIG Nordic

Stockholm
Sweden

Phone +46 (0) 722 25 25 68
e-Mail larsson@mavig.com

Benelux, GB, Ireland
MAVIG B.V.

Mercuriusweg 86
2516AW Den Haag
Netherlands

Phone +31 (0) 70 33 11 688
Mobile +31 (0) 61 595 43 48
e-Mail simmonds@mavig.nl

France
MAVIG France SARL

66, Ave. des Champs Elysées
F-75008 Paris
France

Phone +33 (0)1 30 59 46 23
e-Mail info@mavig.fr

Southeast Europe
MAVIG Southeast Europe

Ljubljana
Slovenia

Phone +386 (40) 6 33 900
e-Mail dejak@mavig.com

REPRESENTATION

USA & Canada
Ti-Ba Enterprises, Inc.

25 Hytec Circle
Rochester, NY 14606
USA

Phone +1 (585) 247 1212
e-Mail mavigusteam@mavig.com

Middle East & Northern Africa
Mena Medical Development

Ashrafieh, Adlieh Square, Alfaras St.,
Alboustany Building, 5th Floor - Beirut
Liban

Phone +961 14 23 499
e-Mail mavig@mena-md.com

Russian Federation
AO Sante Medical Systems

Novodmitrovskaya 2, bld.1
127015 Moscow
Russia

Phone +7 499 551 55 73/75
e-Mail info@sante.ru

www.mavig.com