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RP/RD

USER MANUAL
X-RAY PROTECTION – PATIENT/DENTAL

Languages: **Deutsch** 02-23
 English 27-47

Art.-Code: 1240991

M**A****V****I****G**

MAVIG

www.mavig.com/eifu

EN

- DE** Vielen Dank, dass Sie sich für MAVIG entschieden haben. Die Gebrauchsanweisung finden Sie unter dem Link der Überschrift.
- EN** Thank you for buying MAVIG products. The instruction for use can be found under the link in the headline.
- AR** نشكر على شراء منتجات شركة "مافيج".
يُمكنك إيجاد تعليمات الاستخدام على الرابط في العنوان.
- BG** Благодарим Ви, че избрахте MAVIG. Упътването за употреба може да намерите на линка на заглавието.
- CS** Děkujeme za zakoupení výrobku MAVIG. Návod k použití naleznete na odkazu v titulků.
- DA** Tak fordi du købte MAVIG-produkter. Brugsanvisningen kan findes under linket i overskriften.
- ES** Le damos las gracias por la compra de productos MAVIG. Las instrucciones de uso pueden encontrarse bajo el enlace del titular.
- ET** Täname, et soetasite ettevõtte MAVIG toote. Kasutusjuhend leiata pealkirja lingi alt.
- FI** Kiitos, että valitsit MAVIG-tuotteen. Käyttöohjeet löytyvät otsikon linkin takaa.
- FR** Merci d'avoir acheté les produits de MAVIG. Le mode d'emploi est disponible en cliquant sur le lien qui se trouve au niveau du titre.
- HR** Hvala što ste kupili proizvode društva MAVIG. Uputa za upotrebu nalazi se ispod poveznice u naslovu.
- HU** Köszönjük, hogy MAVIG termékeket vásárolt. A használati utasítások a címsorban lévő hivatkozásokon találhatóak.
- ID** Terima kasih Anda telah membeli produk MAVIG. Petunjuk penggunaan dapat ditemukan di bawah tautan di judul.
- IT** Grazie di aver acquistato i prodotti MAVIG. Le istruzioni per l'uso possono essere consultate sul link del titolo.
- JA** MAVIG製品をお買い上げいただきありがとうございます。使用方法の説明は、見出しのリンクの下に掲載されています。
- KO** MAVIG 제품을 구매해 주셔서 감사합니다. 사용 설명서는 헤드라인의 링크 아래에서 찾을 수 있습니다.
- LT** Dėkojame, kad įsigijote MAVIG gaminius. Naudojimo instrukciją rasite paspaudę nuorodą antraštyje.
- LV** Paldies, ka iegādājāties MAVIG produktus. Instrukciju lietošanai var atrast zem virsraksta saites.
- NL** Hartelijk dank dat u voor MAVIG gekozen heeft. De gebruiksaanwijzing is te vinden onder de link van de titel.
- NO** Takk for ditt kjøp av MAVIG-produkter. Bruksanvisningen finner du under lenken i overskriften.
- PL** Dziękujemy za zakup wyrobów MAVIG. Instrukcję użytkowania można znaleźć pod linkiem w nagłówku.
- PT** Obrigado por adquirir produtos MAVIG. As instruções de utilização encontram-se na ligação presente no título.
- RO** Vă mulțumim pentru achiziționarea produselor MAVIG. Instrucțiunile de utilizare pot fi găsite accesând linkul din antet.
- RU** Благодарим вас за приобретение изделий производства MAVIG. Инструкцию по эксплуатации можно открыть под ссылкой в заголовке.
- SL** Zahvaljujemo se vam za nakup izdelkov MAVIG. Navodila za uporabo so na voljo na povezavi v naslovni vrstici.
- SV** Tack för att du köpt en MAVIG-produkt. Bruksanvisningen kan hittas under länken i rubriken.
- TR** MAVIG ürünlerini satın aldığınız için teşekkür ederiz. Kullanım kılavuzunu başlıktaki linkin altında bulabilirsiniz.
- VI** Cảm ơn bạn đã mua sản phẩm của MAVIG. Bạn có thể xem hướng dẫn sử dụng và sách hướng dẫn lắp đặt ở liên kết trong tiêu đề.
- ZH** 感谢您购买 MAVIG 产品。如需查看使用说明，可点击标题下的链接。

Dear Customer,

Thank you for purchasing a MAVIG product. Please read this document carefully and follow all directions, warnings, and notices.

Feel free to contact us with any questions or comments on our products.

Contact Information

For questions or comments, please refer to the back of this manual for your nearest MAVIG contact.

Disclaimers

The content of this manual is for information purposes only. The products described are subject to change without notice in the course of continuous development. MAVIG only assumes warranty for this manual and the products within the scope of the manufacturer's liability.

Changes

MAVIG products are under continuous development. MAVIG reserves the right to modify the manual, packaging, equipment delivery without prior notice.

Translation

The master document is in German. In case of translation, precedence remains with the German version.

Copyright and Trademarks

All rights reserved. These instructions/manuals are protected by copyright law. MAVIG and Novalite are registered trademarks.

All other trademarks mentioned in the manuals are the property of the corresponding manufacturer.

Table of Contents

1.0	Use and Availability of the Instructions for Use	29
2.0	Warranty	29
3.0	Symbols and Definitions	30
4.0	Intended Use	32
4.1	Reporting Requirements (EU) 2017/745	37
5.0	Specifications	38
5.1	Conditions of Transport & Storage	38
5.2	Identification of the Product	38
5.3	Standards	38
5.4	Product Description	39
5.4.1	Degree of Protection (and Area Weight)	40
5.4.2	Sizes & Dimensions	41
5.5	Replacement Parts	44
5.6	Manufacturer & Contact Information	44
5.7	Repair	44
5.8	Shipment / Return Shipment	44
6.0	Scope of Delivery	44
7.0	Inspection	45
8.0	Cleaning & Disinfecting	45
8.1	Cleaning	45
8.2	Disinfecting	46
9.0	Safe Disposal	47

1.0 Use and Availability of the Instructions for Use

These instructions for use cover the following products: Personal Protective Equipment (PPE)
- X-ray protection for patients/dental and radiation protective covers

Model range	RP270	Mammary gland protection for adults
	RP271	Mammary gland protection for children
	RP274	Protective gonad pants
	RP276	Testes protection
	RP278	Ovary protection
	RP280	Testes protection
	RP643	Protective gonad apron
	RP648	Patient cover for CT
	RP664	Children's radiation protective apron
	RP668	Children's radiation protective coat
	RP679	Ovary protection with belt
	RP689	Patient cover
	RD614	Thyroid and sternum protection for patients
	RD635	Protective apron for dental use
	RD639	Protective allround apron for dental use
	RD642	Protective apron for dental use
	RD644	Protective apron with thyroid protection for dental use
	ST-FS	Radiation protective cover with one cutout
	ST-RZ	Radiation protective cover
	ST-ER	Radiation protective cover with adjustable cutout

This manual is part of the product and therefore should be kept in the immediate vicinity of the product, so that important information and safety instructions for use can be looked up at all times.

Do not give the product to a third party without a valid manual.

2.0 Warranty

Our general terms of business apply. These should be available to the operator latest after completion of the sales contract.

MAVIG guarantees the safety and reliability of the product solely on the following basis:

- The product is used solely for its intended purpose and its operation complies with the information provided in these instructions for use.
- Only original authorised replacement parts or accessories as specified by MAVIG are used. The use of unauthorised parts holds unknown risks and must be avoided at all times.
- No unauthorised changes are made to the product.
- The product is regularly inspected and serviced at the specified times.

The workmanship and materials used have a 24-month manufacturer's warranty.

3.0 Symbols and Definitions

Information in these instructions that is particularly important is highlighted below. Safety warnings are mandatory and must be adhered to in order to prevent personal injury or damage to equipment.

WARNING

Indicates that the information is a warning. Warnings refer to circumstances which could result in injury or death of the patient or operator/user. Read and understand all warnings before using the product.

CAUTION

Indicates that the information is a caution. Cautions advise you of circumstances that could result in damage to the product or improper functioning of the product. Read and understand the cautions before using the product.

WARNING - Magnetic Field

Indicates that the information warns about a magnetic field. Warnings refer to circumstances which could result in injury or death of the patient or operator/user. Read and understand all warnings before using the product.

No Persons with Pacemakers

This symbol indicates that this product must not be used by persons with pacemakers (minimum distance > 20 cm).

NOTE:

Provides usage tips and useful information.

COMPLY WITH MANUAL

This symbol represents to consult accompanying documents.

CE SYMBOL

With this symbol MAVIG declares that the products comply with the relevant regulations set forth in the applicable European Directives.

ARTICLE NUMBER

Shows the manufacturer's article number.

SERIAL NUMBER

Shows the manufacturer's serial number.

BATCH NUMBER

Shows the manufacturer's batch number.

MANUFACTURER

Shows the address and the manufacturer or distributor of the PPE.

DATE OF MANUFACTURE

Indicates the date of manufacture of the product.

3.0 Symbols and Definitions

MEDICAL DEVICE

Indicates that the product is a medical device.

TEMPERATURE LIMIT

Specifies the temperature limit, which the product can be exposed to safely.

STORE PROTECTED FROM SUNLIGHT

Indicates that the product should not be stored in direct sunlight.

KEEP DRY

Indicates that the product should be stored in a dry place.

STORAGE PERIOD

Indicates the maximum storage time in the packaging.

PROTECTION AGAINST IONISING RADIATION

Indicates that the product provides protection against ionising radiation.

DO NOT WASH

Indicates that the product must not be washed in the washing machine.

DO NOT BLEACH

Indicates that the product must not be bleached.

DO NOT IRON

Indicates that the product must not be ironed.

DO NOT TUMBLE DRY

Indicates that the product must not be tumble-dried.


DO NOT DRY CLEAN

Indicates that the product must not be dry-cleaned.

DO NOT WET CLEAN

Indicates that the product must not be wet-cleaned.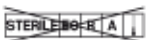

NON-STERILISABLE

Indicates that the product cannot be sterilized with ethylene oxide, radiation, aseptic processing techniques, steam, or dry heat.

4.0 Intended Use

X-ray protection for patients protects against ionising radiation – in the X-ray tube voltage range of 50 - 150 kV (StandardLite) or 50 - 110 kV (NovaLite / OmniLite).

Correct application of the protective products by medical staff is essential. This is essential to ensure optimum radiation protection in accordance with legal requirements.

Inspect products for visible damage before use. For this purpose see also „Inspection“ (Chapter 7.0).

RP270 – Mammary Gland Protection for Adults

Adult mammary gland protection is intended to protect the mammary glands against scattered radiation during mammography recordings.

Hang the protector over the patient's shoulder so one of the protective areas covers the chest and the other hangs on the back of the patient, so that neither protector is in the imaging area. If necessary, adjust the length of the connection with the hook and burr fastener.

RP271 – Mammary Gland Protection for Children

Mammary gland protection for children is intended to protect the mammary glands against scattered radiation during X-rays in medical diagnostics.

Attach the mammary gland protection similar to a bikini top so that it is not in the imaging area. The cord stopper should sit in the neck. Adjust the size of the mammary gland protection using the cord stopper. Then close with a loop on the patient's back. The protective surfaces can be reduced in size with the neoprene stoppers if necessary.

Models RP274, RP276, RP280 – Testes Protection

The testes protection is intended to protect the gonads of the male patient against unnecessary irradiation by the radiation beam and against scattered radiation when the gonads are close to or within the properly limited radiation beam for example during a radiological examination of the pelvis.

RP274 Attach the radiation protection on the patient similar to a diaper. Close the hook and burr fasteners.

RP276 Place or have the patient place the disposable plastic bag over penis and scrotum. Attach or let the patient attach the radiation protection. Close the hook and burr fastener. Using the snap buttons, this model can be unfolded completely for easy cleaning and disinfection.

RP280 Place or have the patient place the disposable plastic bag over penis and scrotum. Attach or let the patient attach the radiation protection.

NOTE: *For hygienic reasons, always use products RP276 and RP280 in combination with a disposable plastic bag for enclosing penis and scrotum.*

In the middle of this manual (page 24) you will find an illustrated customer manual for our products RP276 and RP280. We will gladly send you the manual as a PDF.

Please contact our customer service for this:

Phone +49 (0) 89 / 420 96 - 0

e-Mail info@mavig.com


4.0 Intended Use

Models RP274, RP278, RP679 – Ovary Protection

Ovary protection is intended to protect the ovaries of the patient against unnecessary irradiation by the radiation beam when the ovaries are in the properly limited radiation beam in the antero-posterior projection, for example during radiological examinations of the pelvis.

RP274 Attach the radiation protection on the patient similar to a diaper. Close the hook and burr fasteners.

RP278 Use the ovarian protection RP278 when taking X-rays of patients in supine position. Place the appropriately sized ovarian protection on the patient. With the help of the two lateral radiolucent control marks, the size of the ovarian shield can be precisely adapted to the corresponding pelvis in a fan-like manner. If the control marks are placed on the spinae ilicae anteriores in such a way that the bands form a straight line, the ovarian shield is adapted to the pelvis according to its position and size. (Fig. 1)

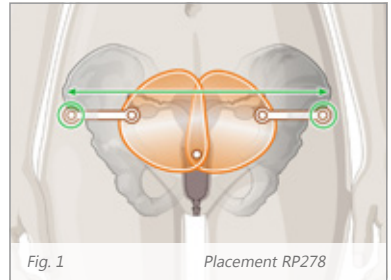RP679 Use the ovarian protection RP679 when taking x-rays of standing patients. Attach the stretch belt with snap buckle to the patient. If necessary, adjust the length of the belt. Attach the appropriately sized protector via the hook and burr fastener provided.

Model RP643 – Protective Gonad Apron

Protective gonad aprons are intended to protect the gonads of patients during radiological examinations of organs other than those located in the area of the lower abdomen, in particular during radiological examinations of the thorax.

Place the tension strap of the appropriately sized gonad apron around the body so that the body part to be protected is covered and the protection is not in the imaging area.

NOTE: *If sufficient coverage is not guaranteed or the strap is too loose, please change the size.*

Model RP648 – Patient Cover for CT

The patient cover is intended to protect the sternum, the breast, and the shoulder joints as well as possibly the thyroid gland against unnecessary irradiation by scattered radiation during CT examinations.

Place the patient cover over the patient so that the sternum, chest, and shoulder joints are covered and the protection is not in the imaging area. If possible, fold the thyroid protection into position.

4.0 Intended Use

Model RP664 – Children’s Radiation Protective Apron

The children’s apron is intended to protect the patient’s torso against unnecessary irradiation by scattered radiation when it is close to the radiation beam.

Put the front apron on the child so that the child’s entire front from the neck to the knees is covered by the apron (radiation protection material) and the protection is not in the imaging area. Close the snap buckle.

NOTE: *If sufficient coverage is not guaranteed, please change the size.*

Model RP668 – Children’s Radiation Protective Coat

The children’s coat is intended to protect the patient’s torso against unnecessary irradiation by scattered radiation when it is iclose to the radiation beam.

Put the coat on the child so that it can be closed in the front. Make sure that the child’s entire front from the neck to the knees is covered by the apron (radiation protection material) and that the protection is not in the imaging area. Close the hook and burr fastener.

NOTE: *If sufficient coverage is not guaranteed, please change the size.*

Model RP689 – Patient Cover

The patient cover is intended to protect the covered body parts against unnecessary irradiation by scattered radiation when the respective body parts are close to the radiation beam.

Place the patient cover on the patient so that the patient’s body parts to be protected are covered and the protection is not in the imaging area. Combine several patient covers as needed.

Models RD635, RD639 – Protection for Dental Use

Dental radiation protective aprons are intended to protect the patient’s chest, upper trunk, and thyroid from scattered radiation during dental radiological examinations of the teeth and the jaw.

RD635 Use patient protection RD635 when taking panoramic X-rays of standing patients. If possible, supplement the model with the thyroid protection RD614.

Put the apron on the patient so that the front of the body is covered from the neck to at least below the gonads as well as the entire sternum and shoulders and the protection is not in the imaging area. Close the hook and burr fastener in the front.

RD639 Use the RD639 patient shield for DVT scans on a standing patients.

Put the apron on the right shoulder of the patient and close the hook and burr on the left shoulder so that the front of the body from the neck to at least below the gonads and the entire sternum and shoulders are covered.

Close the hook and burr closure in the back. Adjust the height of the thyroid shield so that it sits as high as possible but does not interfere with the image.

4.0 Intended Use

Models RD642, RD644, RD614 – Protection for Dental Use

RD642 Use patient protection RD642 when taking X-rays of sitting or supine patients. If possible, supplement the model with the thyroid protection RD614.

Put the apron on the patient so that the front of the body is covered from the neck to at least below the gonads as well as the entire sternum and shoulders and the protection is not in the imaging area. Close the hook and burr fastener in the shoulder area of the patient.

RD644 Use patient protection RD644 with an integrated thyroid protection when taking X-rays of sitting or supine patients. Loops on the back of the apron allow the apron to be attached to the dental chair.

Put the apron on the patient so that the front of the body is covered from the neck to at least below the gonads as well as the entire sternum and shoulders and the protection is not in the imaging area. Close the hook and burr fastener in the neck area of the patient.

RD614 Use the patient thyroid protection to extend the protection of the dental radiation protective aprons RD635 and RD642.

Place the thyroid sternum shield on the patient so that it covers the front of the neck including the thyroid and overlaps the apron. Close the thyroid shield in the neck.

WARNING – Insufficient Protective Area

An overlap of thyroid shield and radiation protection clothing is necessary to ensure continuous radiation protection. Non-continuous coverage with radiation protective clothing increases patient exposure to radiation.

CAUTION

Please note that the magnetic closure of the thyroid shield must be closed in a particular way. Always close the magnetic closure so that - from your point of view - the right magnet is placed over the left magnet. Otherwise, the closure may not hold.

WARNING – Magnetic Field

The magnetic closure of the thyroid shield contains permanent magnets that create a magnetic field. This can lead to malfunctioning of implanted medical devices.

WARNING – No Persons with Pacemakers

The version of the thyroid protection with magnetic closure contains permanent magnets that generate a magnetic field. The thyroid protection is therefore not to be used by medical personnel with pacemakers or on patients with pacemakers.

4.0 Intended Use

Models ST-FS, ST-RZ – Radiation Protective Covers

The radiation protective covers protect the medical staff from scattered radiation emerging from the patient.

Place the correct cover over the groin of the patient so that it is not in the imaging area.

Use ST-FS for unilateral puncture of the femoral artery and ST-RZ for puncture of the radial artery.

NOTE: *For hygienic reasons, always use the products in combination with the respective sterile, disposable cover. Only this allows the radiation protective drapes to be placed above the sterile patient drapes.*

Model ST-ER – Radiation Protective Cover with Adjustable Cutout

The radiation protection cover serves to protect the examiner from out of the patient escaping scattered radiation.

WARNING – Cover in Primary X-Ray Beam

The X-ray protection cover must never be placed inside the area of the primary X-ray beam as this may lead to an increased radiation dose for medical staff, patients and others (fig. 2, 3).

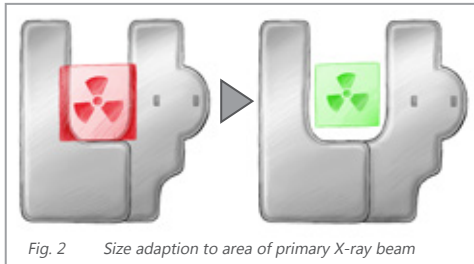
Fig. 2 Size adaption to area of primary X-ray beam

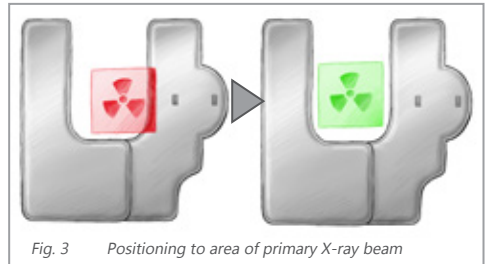
Fig. 3 Positioning to area of primary X-ray beam

Place the cover over the patient without it being in the image-effective area. By placing the radiation protection cover as close as possible to the fluoroscopic area, the radiation exposure of the examiner can be reduced to a minimum.

The X-ray protective cover has been developed for patients lying on the side (Fig. 4) as well as for patients in the prone or supine position (Fig. 5).

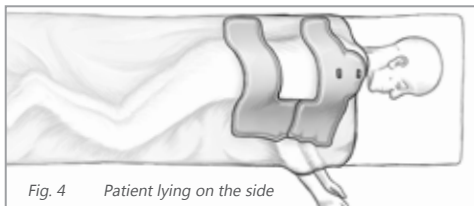
Fig. 4 Patient lying on the side


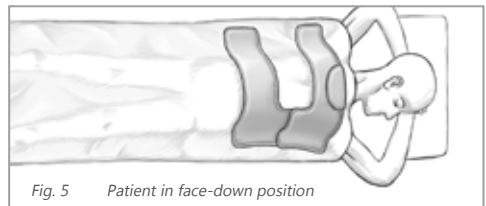
Fig. 5 Patient in face-down position

4.0 Intended Use

Model ST-ER – X-Ray Protective Cover with Adjustable Cutting

The width of the protective cover ST-ER can be adjusted to radiation areas of 15 - 50 cm by using the cut-out with the adaptable opening shown in Fig. 6.

As shown in Fig. 7, the incorporated shoulder protection can be folded over when not needed (depending on the type of examination or positioning of the patient).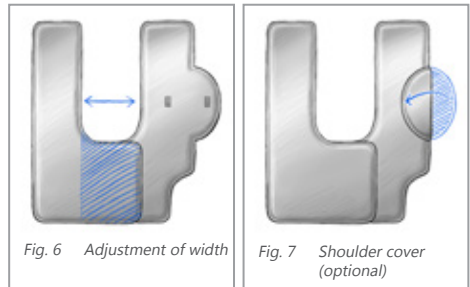


4.1 Reporting Requirements According to Regulation (EU) 2017/745

In accordance with the relevant legal regulations, serious incidents that have occurred in connection with a MAVIG medical device must be reported to MAVIG and to the responsible national authorities in the country in which the user is active. In this sense, incidents are reportable if the use of the medical device directly or indirectly led, might have let or might lead to the death or the temporary or permanent serious deterioration of a patient's, user's or other person's state of health.

5.0 Specifications

5.1 Conditions of Transport & Storage

Follow the storage conditions and the storage period on the packaging.

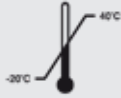Store between
-20°C and + 40°C

Protect
from heat

Protect
from moisture

Maximum
storage duration

Use MAVIG special coat hangers or suitable MAVIG storage systems for storing patient aprons (see MAVIG catalogue). Store all other products on a flat surface. Radiation protection should not be folded or allowed to hang over edges, and must be stored away from sources of heat.

Ensure that radiation protection is kept, stored, and used only within the standard temperature and humidity range for the home and work environment.

5.2 Identification of the Product

The production label contains the following information:

RP270, RP271, RP274, RP648, RP664, RP668, RP679, RP689, RD614, ST-FS, ST-RZ, ST-ER

Model number (REF), description, colour (if applicable), size information (if applicable), material information including tested X-ray tube voltage range, lead equivalent(s), CE marking, serial number, manufacturer, date of manufacture (YYYYMMDD), cleaning/disinfection instruction, care label, and corresponding standards.

RP643, RD635, RD639, RD642, RD644

Model number (REF), description, colour, size information, material information including tested X-ray tube voltage range, lead equivalent(s), CE marking, batch or serial number, manufacturer, date of manufacture (YYYYMMDD), cleaning/disinfection instruction, care label, corresponding standards, and the letter symbol according to the size.

For the meaning of the letter(s) designating the size of the radiation protective apron please refer to „Sizes & Dimensions“ (Chapter 5.4.2).

The radiation protective product contains the following information:

RP276, RP278, RP280

Size information (if applicable), lead equivalent including tested X-ray tube voltage range, CE marking, batch number, manufacturer, date of manufacture (DDMMYY), and corresponding standard.

5.0 Specifications

5.3.1 Standards RP and RD Products

CE marking **CE 0302**

With this symbol MAVIG declares that the products comply with the provisions in the applicable EU directives. Conformity is indicated on the label by the CE mark with the four-digit identification number of the notified body.

MAVIG declares that the products are in conformity with IEC 61331-1:2014, DIN EN 61331-1:2016, IEC 61331-3:2014, DIN EN 61331-3:2016, the German X-ray Ordinance (RöV), the Regulation (EU) 2016/425 of the European Parliament and of the Council of 9 March 2016 on personal protective equipment and repealing Council Directive 89/686/EEC (published in the Official Journal of the European Union, L81, 31 March 2016), and the Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009 and repealing Council Directives 90/385/EEC and 93/42/EEC. MAVIG is DIN EN ISO 13485:2016 certified.

The EC-type examination was conducted by:

Identification number of the notified body: 0302
 ANCCP Certification Agency S.r.l.
 Via dello Struggino, 6
 57121 Livorno LI, Italy

The Declaration(s) of Conformity of personal protective equipment can be found under the following link: www.mavig.com/company/conformity

5.3.2 Standards ST-FS, ST-RZ, and ST-ER Products

CE marking **CE**

With this symbol MAVIG declares that the products comply with the provisions in the applicable EU directives.

MAVIG declares that the products are in conformity with IEC 61331-1:2014, DIN EN 61331-1:2016, and the Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009 and repealing Council Directives 90/385/EEC and 93/42/EEC. MAVIG is DIN EN ISO 13485:2016 certified.

5.4 Product Description

Personal protective equipment – radiation protective clothing – for patients serves to protect against ionizing radiation.

5.0 Specifications

5.4.1 Degree of Protection

The radiation protective materials achieve the following absorption values for the indicated X-ray tube voltages according to IEC 61331-1:2014/DIN EN 61331-1:2016:

Material: Standard Lead										
Weight	Lead equivalent	50 kV	60 kV	70 kV	80 kV	90 kV	100 kV	110 kV	120 kV	150 kV
(3.30 kg/m ²)	0.25 mmPb	99 %	97 %	96 %	93 %	91 %	88 %	87 %	84 %	80 %
(4.60 kg/m ²)	0.35 mmPb	> 99 %	99 %	98 %	96 %	95 %	92 %	92 %	90 %	87 %
(6.60 kg/m ²)	0.50 mmPb	> 99 %	> 99 %	99 %	98 %	97 %	96 %	95 %	94 %	92 %
(13.2 kg/m ²)	1.00 mmPb	> 99 %	> 99 %	> 99 %	> 99 %	99 %	99 %	99 %	99 %	98 %

Material: Novalite							
Weight	Lead equivalent	50 kV	70 kV	90 kV	110 kV	150 kV	
(3.05 kg/m ²)	0.25 mmPb	99 %	96 %	91 %	87 %	-	
(4.30 kg/m ²)	0.35 mmPb	> 99 %	98 %	95 %	92 %	-	
(6.10 kg/m ²)	0.50 mmPb	> 99 %	99 %	97 %	96 %	-	

Material: Leadfree							
Weight	Lead equivalent	50 kV	70 kV	90 kV	110 kV	150 kV	
(2.95 kg/m ²)	0.25 mmPb	99 %	96 %	91 %	85 %	-	
(4.18 kg/m ²)	0.35 mmPb	> 99 %	98 %	94 %	91 %	-	
(5.89 kg/m ²)	0.50 mmPb	> 99 %	99 %	97 %	95 %	-	

5.0 Specifications

5.4.2 Sizes & Dimensions

CAUTION – The dimensions of the protective area are not exactly equal to the specified dimensions.

Please note that the protective area (area without pinholes) ends at the seams. This also applies to e.g. the thyroid shield of the dental apron RD644. Therefore, the dimensions of the protective area (if not specifically mentioned) are not exactly equal to the specified dimensions.

Product dimensions are subject to a tolerance of +/- 1 cm.

RP270 – Mammary Gland Protection, for adults

Size	Length	Width
One size fits all	40 cm	18 cm

RP271 – Mammary Gland Protection, for children

Size	Length	Width
One size fits all	11 cm	10 cm

RP274 – Protective Gonad Pants

The set for children contains a testes protection for infants, toddlers, and children from 1 to 6 years of age.

RP276 – Testes Protection

Size	Length	Width	Depth
E (adults)	15.5 cm	12.0 cm	9.0 cm
K (children)	11.0 cm	9.0 cm	6.5 cm
G (toddlers/set)	The set for toddlers contains a testes protection for infants, toddlers, and children from 1 to 10 years of age.		

RP280 – Testes Protection

Size	Length	Width	Depth
E (adults)	10.5 cm	13.5 cm	6.0 cm
K (children)	8.5 cm	10.5 cm	5.5 cm

RP278 – Ovary Protection

Size	Length	Width
S (small)	7 cm	variable
M (medium)	9 cm	variable
L (large)	12 cm	variable

5.0 Specifications

5.4.2 Sizes & Dimensions

RP679 – Ovary Protection, Set E for adults

Size	Length	Width
S (small)	16 cm	11 cm
M (medium)	18 cm	13 cm
L (large)	20 cm	19 cm

RP679 – Ovary Protection, Set K for children

Size	Length	Width
S (small)	7 cm	6 cm
M (medium)	9 cm	8 cm
L (large)	10 cm	10 cm

RP679 – Ovary Protection, for premature babies

Size	Length	Width
One size fits all	4 cm	3 cm

RP643 – Protective Gonad Apron

Size	Symbol*	Recommended	Length	Width
S (small)	C1	for Toddlers	20 cm	25 cm
M (medium)	C2	for Children	30 cm	30 cm
L (large)	A1	for Adults	37 cm	40 cm
X (x-large)	A2	for Adults	45 cm	50 cm

* according to IEC 61331-3:2014 / DIN EN 61331-3:2016

RP648 – Patient Cover for CT

Size	Length	Width
One size fits all	35 cm	63 cm

RP689 – Patient Cover

Size = Customer specific dimensions

RP664 – Radiation Protective Apron, for Children

Size	Recommended, Age	Length	Width
S (small)	for Children, ca. 3 - 5 years	59 cm	35 cm
M (medium)	for Children, ca. 6 - 8 years	67 cm	43 cm
L (large)	for Children, ca. 9 - 12 years	84 cm	45 cm

5.0 Specifications

5.4.2 Sizes & Dimensions

RP668 – Radiation Protective Coat for Children

Size	Recommended, Age	Length
<i>S (small)</i>	for Children, ca. 3 - 5 years (Hip girth ca. 68 - 75 cm)	60 cm
<i>M (medium)</i>	for Children, ca. 6 - 8 years (Hip girth ca. 76 - 83 cm)	80 cm
<i>L (large)</i>	for Children, ca. 9 - 12 years (Hip girth ca. 84 - 95 cm)	84 cm

RD635, RD639, RD642, RD644 – Dental Protection

	Size	Symbol*	Length	Width
	<i>E (adults)</i>	DA1	80 cm	60 cm
	<i>K (children)</i>	DC1	60 cm	45 cm

* according to IEC 61331-3:2014 / DIN EN 61331-3:2016

RD614 – Dental Thyroid Protection

Size	Closure	Neck Sizes
<i>E (adults)</i>	Hook and burr	32 - 45 cm
<i>E (adults)</i>	Magnet	32 - 45 cm
<i>E (adults)</i>	Replaceable hook/burr	37 - 50 cm
<i>K (children)</i>	Hook and burr	28 - 36 cm
<i>K (children)</i>	Magnet	28 - 36 cm
<i>K (children)</i>	Replaceable hook/burr	28 - 36 cm

5.0 Specifications

5.5 Replacement Parts

For an up-to-date list of replacement parts, please contact us at:

Phone +49 (0) 89 / 420 96 - 0 e-Mail info@mavig.com

5.6 Manufacturer & Contact Information

MAVIG GmbH	Phone	+49 (0) 89 / 420 96 - 0
Stahlgruberring 5	Fax	+49 (0) 89 / 420 96 - 200
81829 Munich	e-Mail	info@mavig.com
Germany	Web	www.mavig.com

5.7 Repair

A repair service is available through our factory in Munich. Our service hotline will gladly provide you with all the information needed (see Chapter 7.0 "Inspection").

We do not accept any warranty for repairs not performed by MAVIG.

5.8 Shipment / Return Shipment

MAVIG pradiation protection must not be subjected to mechanical stress during shipment. It is not allowed to fold, bent, or stack radiation protection. It is recommended that the products are loosely rolled and packed properly supported in a strong shipping box of a suitable size.

To return goods to our factory in Munich, please contact our service hotline in advance (see Chapter 5.6 "Manufacturer and Contact Information"), so that we can send you the necessary return documents (including decontamination certificate).

NOTE: *It is prohibited by law to send potentially contaminated goods. Any returns must therefore be properly cleaned and decontaminated. Please attach the completed decontamination certificate on the outside of the packaging.*

6.0 Scope of Delivery

Each radiation protective product comes with an instruction for use.

RP664, RP668, RD635, and RD644 also include a MAVIG special hanger that can be used to safely store the radiation protective apron.

7.0 Inspection

CAUTION

Prior to each use of the product, check for visible damage.
Do not use a damaged product!

To ensure the radiation protection still provides complete protection, it should be checked regularly, at least once every 3 months. Tears, clearly visible kinks, protrusions, and uneven areas are all indications of a defect.

We recommend that the radiation protective clothing is inspected in the style of the standard DIN 6857-2:2021. This involves a fluoroscopic examination with X-rays at regular intervals, in addition to regular visual and tactile tests.

Please contact us if you suspect a defect:

Phone +49 (0) 89 / 420 96 - 0
Fax +49 (0) 89 / 420 96 - 200
e-Mail info@mavig.com

8.0 Cleaning & Disinfecting

NOTE:

*Lacking hygiene can lead to infections of patients, operators/users, or third parties.
Therefore clean and disinfect according to the applicable statutory provisions.*

The following markings apply to the radiation protective products:

Do not
wash

Do not
bleach

Do not
iron


Do not
tumble dry

Do not
dry clean

Do not
wet clean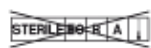

Non-sterilisable

8.1 Cleaning

WARNING – Damage the Product

In order to avoid damage to the product, no aggressive, abrasive or corrosive agents should be used (e.g. diluent, acetone, scouring agents).

RP270, RP271, RP274, RP276 (set size G), **RP643, RP648, RP664, RP668, RP679, RP689, RD614, RD635, RD639, RD642, RD644, ST-FS, ST-RZ, ST-ER:**

Fresh stains can usually be removed from the radiation protective clothing with soapy water of approx. 20°C and a soft cloth. For stubborn stains, we recommend a 10% solution of sodium thiosulphate in water. For fluids containing proteins (such as blood), use cold water. Please make sure that the cloth is damp and not wet.

RP278, RP276 (sizes E and K), **RP280:**

Remove light stains with lukewarm soapy water and a soft cloth. For stubborn stains, remove spots with a linen cloth dipped in alcohol and rinse with clear water.

8.0 Cleaning & Disinfecting

8.2 Disinfection

The product should always be cleaned as described in Chapter 8.1 „Cleaning“ prior to disinfection.

WARNING – Product Performance and Surface Damage

The combined use of different disinfectants and/or disinfectants with cleaning agents may impair the product performance or damage the surfaces of the product.

RP270, RP271, RP274, RP276 (set size G), RP643, RP648, RP664, RP668, RP679, RP689, RD614, RD635, RD639, RD642, RD644, ST-FS, ST-RZ, ST-ER:

WARNUNG – Damage the Product

Do not use alcohol-based disinfectants, pure alcohol (e.g. ethanol and isopropanol), or oxidising as well as active chlorine or salt-containing disinfectants as they attack the surface of the outer material more strongly than disinfectants based on other chemicals.

Avoid applying too much pressure while disinfecting. To maintain the properties of our radiation protective clothing, we recommend the use of the following disinfectants. Please observe the instructions below:

- The tests carried out at our company do not correspond to any certified or standardized test procedures, so that we cannot issue a release, but only a recommendation.
- Since we have no influence on the production as well as the raw materials of the disinfectant, we have to decline any guarantee on the resistance of our protective clothes to the disinfectant.
- The effectiveness of the surface disinfectant has not been considered. The instructions for use of the disinfectant must be adhered to.
- For the reasons mentioned above MAVIG excludes any warranty or liability for subsequent damages.

Disinfectant	Manufacturer	Effective base
Microbac forte	BODE Chemie GmbH	Quaternary compounds, alkylamine derivative
Kohrsolin FF	BODE Chemie GmbH	Aldehyde, quaternary compounds
Antifect FF	Schülke & Mayr GmbH	Quaternary compound, aldehyde
Microzid Sensitive Wipes	Schülke & Mayr GmbH	Quaternary compounds
Microbac Tissues Solution	BODE Chemie	Quaternary compounds

In general, when choosing a disinfectant, make sure it contains as few alcohols as possible. Ideally, the first 5 ingredients should not be alcohols or these should be listed with amounts of „< 5%“.

If you have any further questions about disinfectants or if your clinic does not allow the disinfectants listed above, please do not hesitate to contact us:

Phone +49 (0) 89 / 420 96 - 0

e-Mail info@mavig.com

8.0 Cleaning & Disinfecting

8.2 Disinfection

RP278, RP276 (sizes E and K), **RP280**:

Use disinfectants that are suitable for PVC or PU, and which are normally used in medicine. Please rinse with clear water. Avoid applying too much pressure when using disinfectant wipes.

9.0 Safe Disposal

The product must be disposed of in accordance with the nationally applicable provisions at an appropriate recycling facility.

If you have any questions please do not hesitate to contact us at/on:

Phone +49 (0) 89 / 420 96 - 0
e-Mail info@mavig.com

MAVIG GmbH
Headquarters

PO Box 82 03 62
81803 Munich
Germany

Stahlgruberring 5
81829 Munich
Germany

Phone +49 (0) 89 420 96 0
Fax +49 (0) 89 420 96 200
e-Mail info@mavig.com

WORLDWIDE MAVIG OFFICES

Nordic & Baltic Countries
MAVIG Nordic

Stockholm
Sweden

Phone +46 (0) 722 25 25 68
e-Mail larsson@mavig.com

Benelux, GB, Ireland
MAVIG B.V.

Mercuriusweg 86
2516AW Den Haag
Netherlands

Phone +31 (0) 70 33 11 688
Mobile +31 (0) 61 595 43 48
e-Mail simmonds@mavig.nl

France
MAVIG France SARL

66, Ave. des Champs Elysées
F-75008 Paris
France

Phone +33 (0)1 30 59 46 23
e-Mail info@mavig.fr

Southeast Europe
MAVIG Southeast Europe

Ljubljana
Slovenia

Phone +386 (40) 6 33 900
e-Mail dejak@mavig.com

REPRESENTATION

USA & Canada
Ti-Ba Enterprises, Inc.

25 Hytec Circle
Rochester, NY 14606
USA

Phone +1 (585) 247 1212
e-Mail mavigusteam@mavig.com

Middle East & Northern Africa
Mena Medical Development

Ashrafieh, Adlieh Square, Alfaras St.,
Alboustany Building, 5th Floor - Beirut
Liban

Phone +961 14 23 499
e-Mail mavig@mena-md.com

Russian Federation
AO Sante Medical Systems

Novodmitrovskaya 2, bld.1
127015 Moscow
Russia

Phone +7 499 551 55 73/75
e-Mail info@sante.ru

www.mavig.com