Produktgruppe
Product Group

UT – Röntgenschutz Unterkörper
UT – Radiation Lower Body Protection

Produkte/Products

UT58-99

DISCLAIMER

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DISCLAIMER

Dear Customer,

Thank you for choosing a MAVIG product. Please read this document carefully and follow all safety relevant instructions and notes.

The manual is part of the product and must therefore be kept in a place close to it so that important information and safety instructions for use can be looked up at any time.

Do not pass the product on to third parties without the valid instructions for use.

Please contact us if you have questions or comments about our products.

Contact information

If you have any questions or comments, you will find contact information for your local MAVIG subsidiary at the end of this manual.

Disclaimers

The content of this manual is for information purposes only. The products described are subject to change without notice in the course of continuous development.

MAVIG only assumes warranty for this manual and the products within the scope of the manufacturer's liability.

MAVIG is not responsible for unauthorized modifications made to the product or use of the product for unintended purposes. MAVIG cannot be held liable for improper operation and modifications. Since improper modifications may affect the correct operation, safety or reliability of the product, product modifications require written authorization from MAVIG.

Changes

MAVIG products are constantly being further developed. MAVIG reserves the right to change the manual, packaging, equipment or technical specifications as well as scope of delivery without notice.

Translations

The original document is in German. In the event of a translation, the German version shall remain the authoritative version.

Copyright and trademarks

All rights reserved. These instruction is protected by copyright. MAVIG is a registered trademark. All other trademarks mentioned in these instruction are the property of the respective manufacturer.

INDEX

1.	General Information	12
1.1.	Manufacturer	12
1.2.	Operation and Operator	12
1.3.	Symbols, Safety and Warning instructions	12
1.4.	Reporting Requirements according to Regulation (EU) 2017/745	12
2.	Product Information	13
2.1.	Product Description	13
2.2.	Intended Use	13
2.3.	Contraindications and Exclusions	13
2.4.	Identification	13
2.5.	CE-Marking	13
2.6.	Accessories and Spare Parts	13
3.	Installation and Use	14
3.1.	User Environment	14
3.2.	Installation, Use	14
3.2.1.	Installation	14
3.2.2.	Use	14
3.3.	Cleaning, Disinfection, Sterilization	15
3.3.1.	Cleaning	15
3.3.2.	Disinfection	15
3.4.	Maintenance and Repair	15
3.5.	Calibration and Verification	15
3.6.	Warranty	15
4.	Storage, Transport, Shelf Life	17
5.	Technical Data	17
6.	Decommissioning and Disposal	17

1. General Information

1.1. Manufacturer	MAVIG GmbH Stahlgruberring 5 81829 Munich Germany
Phone	+49 (0) 89 420 96 - 0
Fax	+49 (0) 89 420 96 - 200
e-mail	<i>info@mavig.com</i>
Web	<i>www.mavig.com</i>

1.2. Operation and Operator

Only appropriately trained personnel may operate, clean and disinfect the product (medical and radiological technical knowledge in accordance with country specific regulations).

The instructions for use are valid only after a properly conducted first commissioning.

Product must be thoroughly cleaned and disinfected before first use.

1.3. Symbols, Safety and Warning instructions

SAFETY-RELEVANT	Safety relevant instructions are mandatory and must be followed. Non-observance could result in injury or death of one or more persons. You must read and understand these instructions before starting up the systems.
CAUTION	Cautions must be observed, otherwise the device may be damaged or malfunction . You must read and understand these instructions before starting up the system.

1.4. Reporting Requirements according to Regulation (EU) 2017/745

Since the products are not a medical device in the sense of the EU Medical Device Regulation 2017/745 (MDR), there are also no statutory reporting obligations according to the EU Medical Device Regulation 2017/745 (MDR).

2. Product Information

2.1. Product Description

The lower body scattered radiation protection is used to protect medical staff against ionising radiation in the X-ray tube voltage range of 50 – 150 kV. The product includes a table padding without radiation protection and individual X-ray protective panels which are secured to the table padding by means of Velcro fixture.

UT58-99 does not require any fastening options at the table and can therefore be secured to any table with appropriate dimensions. The number and length of the panels as well as the dimensions of the table padding with Velcro fixture are adjusted to each table according to customer requirements.

Article Code	Width of the protective area of a panel [m]	Lead equivalent [mmPb]	Mass per unit area [kg/m ²]
UT58-99	0.211	0.500	6,900 – 7,700

The weight of the lower body protective system is mainly composed of the maximum weight of the radiation protection material which can be calculated with the following formula:

Max. weight of protective material [kg] = max. mass per unit area [kg/m²] * panel width (protective area) [m] * panel length (protective area) [m] * number of panels

2.2. Intended Use

Radiation lower body protections are used for the stationary protection of medical personnel against ionizing radiation. The products are designed for use in air-conditioned healthcare facilities such as clinics, hospitals, and doctor's practices. The products are used exclusively by medical professionals such as physicians and assistants.

2.3. Contraindications and Exclusions

Any other purpose than the one described is not intended. The manufacturer is not liable for any resulting damage.

2.4. Identification

The product can be clearly identified by using the type plate/product label attached on the product. There you will find at least the following information:

- manufacturer and address
- serial number

2.5. CE-Marking

This product does not have the CE mark.

2.6. Accessories and Spare Parts

Only the original manufacturer's components and spare parts must be used with respect to this product. Failure to comply with this instruction will exempt the manufacturer from any liability or warranty obligations. The operator undertakes to ensure that the device is used only with permitted replacement parts. The use of unauthorized parts entails unknown risks and must be avoided at all times.

For information about available spare parts and accessories, please refer to our data sheets, which are available at our website www.mavig.com or contact us at:

e-mail info@mavig.com

Phone +49 (0) 89 420 96 – 0

SAFETY-RELEVANT

Use of spare parts from third-parties can impact the safety and result in damage, incorrect function or total failure of the product. Only use original spare parts!

3. Installation and Use

3.1. User Environment

During the use of this product, the room temperature and humidity should remain in a range for normal working and living conditions.

Do not keep the lower body protection in close proximity to heat sources.

3.2. Installation, Use

3.2.1. Installation

CAUTION	Please observe the maximum additional load permitted for the table which is provided in the manufacturer's specifications.
----------------	--

For assembly, first place the table padding, which does not include any radiation protection material, onto the table with the Velcro facing upwards. Make sure that the table padding is not wider than the table itself.

It is essential that the lower body protection is correctly mounted to the table. Only if this requirement is met, optimum radiation protection according to the legal provisions can be ensured.

Make sure that the table padding is not wider than the table itself.

SAFETY-RELEVANT	<p>Insufficient protective area</p> <p>If the table padding is wider than the table, there is no radiation protection between the end of the protective panels and the edge of the table after having applied the panels. This will result in increased radiation exposure.</p>
------------------------	--

Now apply the panels to the Velcro. Make sure that the hook side and the soft side of the Velcro have maximum overlap. Make sure that the panels overlap at least by 3 cm. The panels should overlap along their complete length.

SAFETY-RELEVANT	<p>Insufficient protective area</p> <p>Any insufficient overlap of the panels may cause insufficient radiation protection. This will result in increased radiation exposure.</p>
------------------------	---

3.2.2. Use

The radiation protection may neither be deposited on contaminated surfaces nor placed or pulled over sharp edges. Do not bend the panels.

3.3. Cleaning, Disinfection, Sterilization

3.3.1. Cleaning

For wiping all surfaces:

- Use soft, particle-free and free of residues of other cleaning agents cloths for cleaning.
- Only slightly moisten the cloth. If necessary, use a mild soap solution or commercial washing-up liquid.
- Make sure that no liquid is spilled during cleaning and penetrates the device.
- Then rub dry the surfaces with a cloth.

CAUTION	<p>Damage to the product</p> <p>In order to avoid damage to the product no aggressive, abrasive, caustic, bleaching, alkaline, acidic or corrosive agents should be used (e.g. solvents, acetates, scouring agents, chlorine etc.).</p>
----------------	--

3.3.2. Disinfection

Only use wipe disinfection.

- No liquid may penetrate the product.
- The applied reprocessing process must comply with the applicable legal regulations and guidelines.
- Hygiene procedures and conditions are the responsibility of the operator and must be determined by him.
- Ensure that the materials used are compatible (e.g. stainless steel, aluminum, plastic, lead acrylic, lead glass, PVC, natural, silicone or synthetic rubber, chromed, painted or coated surfaces).
- In case of doubt, test the product suitability in an inconspicuous place.

CAUTION	<p>Surface Damage</p> <p>The combined use of different disinfectants and/or disinfectants and cleaning agents may impair the product performance or damage surfaces of the product.</p>
----------------	--

3.4. Maintenance and Repair

MAVIG GmbH distributes its products through authorized dealers only. Those dealers possess the necessary knowledge for servicing, maintenance and repair of the product. Please approach your authorized dealer first in case you need any of the above mentioned services.

3.5 Calibration and Verification

CAUTION	<p>Check for visible damage prior to usage of product. Do not use a damaged product!</p>
----------------	--

3.6. Warranty

Our general terms of business apply. These should be available to the operator latest after completion of the sales contract.

MAVIG guarantees the safety and reliability of the product/ system solely on the following basis:

- The product/system is used solely for its intended purpose and its use complies with the information provided in these instructions for use.
- Only original authorised replacement parts or accessories as specified by MAVIG are used. The use of unauthorised parts holds unknown risks and must be avoided at all times.
- No unauthorised structural or other changes are to be made to the product/system. Unauthorised changes or modifications to the product are prohibited for safety reasons.
- The product/system is regularly inspected and serviced at the specified times.

The manufacturer's warranty period for workmanship and materials is 24 months.

4. Storage, Transport, Shelf Life

Stored only in closed or covered spaces.

Transport and storage conditions:

- Temperature: -20° to +70°C (-4 to 158°F)
- Relative humidity: 0 to max. 95% rel. humidity (non-condensing)
- Air pressure: 70 bis 106 kPa

In service:

- Temperature: +10° bis +40°C
- Relative humidity: 20 to max. 75% rel. humidity
- Air pressure: 70 bis 106 kPa.

5. Technical Data

Please inform yourself about the technical performance of the product in our product data sheets and product catalogs, which are published on our website www.mavig.com or contact us at:

e-mail	info@mavig.com	Phone	+49 (0) 89 420 96 - 0
--------	--	-------	-----------------------

6. Decommissioning and Disposal

The device must be taken out of operation if it is damaged or has not passed the maintenance test.

The device must be disposed of in accordance with your country-specific regulations.

Please feel free to contact us if you have any questions:

e-mail	info@mavig.com	Phone	+49 (0) 89 420 96 - 0
--------	--	-------	-----------------------

WORLDWIDE MAVIG OFFICES

REPRESENTATION

MAVIG GmbH
Headquarters

PO Box 82 03 62
81803 Munich
Germany

Stahlgruberring 5
81829 Munich
Germany

Phone +49 (0) 89 420 96 0
Fax +49 (0) 89 420 96 200
e-Mail info@mavig.com

Nordic & Baltic Countries
MAVIG Nordic

Stockholm
Sweden

Phone +46 (0) 722 25 25 68
e-Mail larsson@mavig.com

Benelux, GB, Ireland
MAVIG B.V.

Mercuriusweg 86
2516 AW Den Haag
Netherlands

Phone +31 (0) 70 33 11 688
Mobile +31 (0) 61 595 43 48
e-Mail simmonds@mavig.nl

France
MAVIG France SARL

66, Ave. des Champs Elysées
F-75008 Paris
France

Phone +33 (0)1 30 59 46 23
Fax +33 (0)1 30 59 46 23
e-Mail info@mavig.fr

Southeast Europe
MAVIG Southeast Europe

Ljubljana
Slovenia

Phone +386 (40) 6 33 900
e-Mail dejak@mavig.com

USA & Canada
Ti-Ba Enterprises, Inc.

25 HYTEC Circle
Rochester, NY 14606
USA

Phone +1 (585) 247 1212
Fax +1 (585) 247 1305
e-Mail mavigusa@mavig.com

Middle East & Northern Africa
Mena Medical Development

Ashrafieh – Adieh Square – Alfaras St.
- Alboustany Building, 5th floor
Beirut
Lebanon

Phone +961 14 23 499
Fax +961 14 26 499
e-Mail mavig@mena-md.com

Russian Federation
AO Sante Medical Systems

Novodmitrovskaya 2, bld.1
127015 Moscow
Russia

Phone +7 499 551 55 73/75
e-Mail info@sante.ru

www.mavig.com